



Dental Design Studio Inc.

2312 Bethards Drive, Santa Rosa CA 95405

phone: (707) 568-7680 fax: (707) 568-7667 toll free: (800) 568-9688

email: dds1lab@aol.com website: <http://www.dds1abs.com>

Thank you for your splint order. Manufacturer's instructions for patient care are as follows:

Do not store in water – soak for no more than 10 minutes in warm tap water.

Do not clean with any alcohol based products.

Do not use standard toothpaste.

Use soap and water for cleaning or non-abrasive denture cream.