


# Sterilization best practice quick guide (Canada)

Each Load
 Each Day

## CSR Sterilization Wrap

- Highest BFE (bacterial filtration efficiency) available\*
- Highly permeable (no wet packs)
- Made from environmentally-friendly cellulose
- Exceptional tensile strength (wet or dry)
- Use with EtO or steam sterilization processes
- Recommended use of two layers folded sequentially

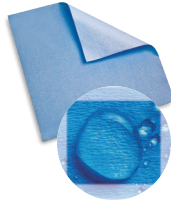

\*Data on File

OR

## Sure-Check™ Type 4 Sterilization Pouches

Built-in multi-parameter internal/external process indicators that change color when all three criteria for sterilization have been met – time, temperature and steam.



## STEAMPlus™ Sterilizer Test Pack (one time per load)

PCD with Type 5  
Allows users to immediately release non-implant loads with peace of mind from the STEAMPlus™ Sterilization Integrator, which is attached to a load record card.



OR

## STEAMPlus™ Type 5 Integrators (one time per pack)

Type 5  
Allow users to immediately release non-implant loads with peace of mind from the STEAMPlus™ Sterilization Integrator.



Steam Indicator Tape made outside the US.

## SporView™ Plus Biological Indicator Test Pack

PCD with BI + Type 5  
Allows users to immediately release non-implant loads with peace of mind from the SporView™ Plus Sterilization Integrator, which is attached to a load record card.



OR

## SporView™ Self-Contained Biological Indicator

An in-office biological indicator (BI) system certified for use with all steam sterilization cycles.

This system gives clear, easy to interpret results for monitoring the effectiveness of the sterilization process.



Biological indicator results are known after 24 hours incubation in any compatible dry block incubator operating at 55°C to 60°C.



## AirView™ II Bowie-Dick Test Pack (for pre-vac sterilizers)

- Designed to detect the presence of air in prevacuum steam sterilizers
- Available in single-use
- Easy-to-interpret indicator sheet
- Ink will not fade after sterilization
- ISO 11140-5 Type 2 Compliant



Implant loads must be held until BI results are known.



All product names are trademarks of Crosstex International Inc., a Cantel Medical Company, its' affiliates or related companies, unless otherwise noted.  
© 2019 Crosstex International, Inc. DLIT00915 Rev A 0519



Tel (519) 681-4763  
Fax (519) 681-9317  
1-866-833-9352  
Email: quality@qfootcare.com  
www.qualitypodiatryproducts.com