



# ***Dental Design Studio, Inc***

2312 Bethards Drive, Suite 4 Santa Rosa, CA 95405  
phone (707)568-7680 fax (707)568-7667  
toll free (800)568-9688

email: [dds1lab@aol.com](mailto:dds1lab@aol.com) website: [www.ddslabs.com](http://www.ddslabs.com)

***Thank you for your TCS / Duraflex order!***

***Manufacturer's instructions for patient insertion and care are as follows:***

After receiving the case, and immediately before placing in patients mouth, immerse the partial in very hot tap water. Leave the partial in the water for up to 1 minute. Remove and allow to cool to the temperature the patient is comfortable with. Place in the mouth. The hot water allows a comfortable placement in the mouth. If it feels too tight because of a tight clasp, the clasp can be loosened slightly by placing the partial back into the hot water and bending the clasp outwardly. If a clasp needs to be tightened, place in the hot water and bend inward. To keep the clasp strength, DO NOT adjust the thickness in the clasp area.

Do not store in water – soak for no more than 1 minute in warm tap water before wearing.

Do not clean with any alcohol based products.

Do not use standard toothpaste.

Use soap and water for cleaning or non-abrasive denture cream.