

MiniBuddy warranty information:

General warranty: One Year limited warranty from date of purchase.

ALDS warrants the **MiniBuddy** amplifier against defects in materials and/or workmanship for a period of 12 months from the date of purchase.

Accessories such as the following have a 6 month warranty from date of purchase as a kit with the **MiniBuddy**:
Microphones, charge cables, charger (if supplied with the original sale) belt clip and carry case.

Exclusions: The warranty does not cover damage to or failure of the unit or accessories due to heat, moisture, shock, misuse, poor maintenance, improper charging, unusual wear and tear or tampering.

Warranty repair or replacement: In the event of failure, the **MiniBuddy** and/or accessories will be repaired or replaced at the manufacturers discretion.

Warranty Procedure:

All warranties are handled directly by the manufacturer.

To initiate a warranty claim, the procedure is as follows:

Contact ALDS warranty department on +1.866.845.2537 or email info@alds.com to receive a returns goods authorization number.

Send the item(s) back to ALDS with your supplied returns goods authorization along with a copy of the original bill of purchase.

Include with the package a brief description of the problem.

The item(s) must be sent back to the warranty depot freight pre-paid and in a safely packaged method. It is advisable to use a shipping method which provides you with a tracking number of your shipment.

Include your full contact information such as name, phone number, email, preferred method of contact and return shipping address.

Please Note: Any items returned to the warranty repair depot without a returns goods authorization will be refused. Any items returned collect will be refused. Any items returned without a copy of the original bill of purchase will be refused.

Your MiniBuddy was distributed by:

MiniBuddy



ALDS
MiniBuddy

Instruction Manual

Congratulations on your purchase of an **ALDS MiniBuddy**

The **ALDS MiniBuddy voice amplifier** has been designed for those with temporary or permanent voice impairment. When using the **MiniBuddy**, a comfortable speech level can be achieved for effective everyday communications while reducing vocal stress and strain.

The **MiniBuddy** can boost vocal output from a soft whisper to a normal level speaking volume when used with the ALDS headset boom microphone (652-HM100NI) or ear hook style microphones+.

The **MiniBuddy** features:

- Patented stylish design and appearance
- Ergonomic & comfortable design
- Lithium light weight rechargeable battery
- Extended battery life
- Powerful double magnetic trumpet for cleaner penetrating sound
- 10 watts of powerful output with excellent sound coverage
- Superior MP3 audio encoding technology supporting TF card & U-flash drive disk

Outpower	10W	Frequency response	90Hz - 18KHz
Output Impedance	4 ohm	Work Temperature	-10C to +45C
Charging voltage	DC 5V 500mA	Amplification time*	8-12 hours
Music play time	6-10 hours	Charging time	3-5 hours
		* at 50% volume for extended periods	

ALDS **MiniBuddy** Complete Kit:

- **MiniBuddy** Amplifier
- 652-HM100NI advanced technology microphone
- Audio cable for connection to other audio sources.
- Mini-USB charging cable
- Charge plug
- Carry case
- Carry case / waist band strap
- Integrated belt clip



+ contact your distributor to find out more information on microphone options for the **MiniBuddy**

1. **Power switch & volume control:** Rotate the power wheel to turn on or adjust the amplifier volume. When the amplifier is turned on the blue light (8) will be continuously on.
2. **Mode:** short press: Short press the mode button to select a user mode (USB, TF). When a USB or TF card is inserted, press the mode button to cycle to the storage card you wish to play.
3. **Play/Pause:** A short press of the play/pause button will play or pause the sound effects from the USB or TF cards. A long press of the button till the blue light (8) blinks quickly will cause the playback to continuous cycle the current sound track.
4. **Previous:** A short press of the button to go to previous sound effects or a long press to fast backwards to initial starting point.
5. **Next:** Short press of the button to go to next sound effect track. A long press will enable fast forwarding.
6. **TF card slot:** Insert a TF card to play MP3 files. The **MiniBuddy** is compatible with most brands to a max capacity of 32GB.
7. **3.5MM microphone jack port**
8. **Indicator light:** Blue light will be on all the time in the state of power on and in use. Red Light will blink when charging the unit. The light will be solid red when **MiniBuddy** is fully charged
9. **USB port:** Insert a USB to play MP3 files, The **MiniBuddy** is compatible with most brands to a max capacity of 32GB
10. **Mini-USB charge cable port (DC 5V):** Use only the supplied charge cable for charging the **MiniBuddy**. IMPORTANT – connect the charge cable with a compatible USB charger of DC 5V 9 500mA – 1.5A) charger. Unplug the **MiniBuddy** as soon as it is fully charged to maintain safety and battery integrity.
11. **Detachable belt clip**

